

The new
Cutera® Titan™
gives you an

all-natural option to
enhance skin tone. Titan
uses light energy to heat
the dermis deep beneath
your skin's surface.

Without surgery,
recovery time, or
injections, Titan is an ideal
solution for your face,
arms and abdomen.



SKIN
TIGHT

CUTERA™

SAN FRANCISCO HEADQUARTERS
3240 Bayshore Boulevard, Brisbane, CA 94005
Phone: 1-888-4CUTERA | www.cutera.com

© 2005 Cutera, Inc. All rights reserved. CoolGlide and Cutera are registered trademarks of Cutera, Inc.

PB0006a

CUTERA™

Skin Tight in a New Light

How does the Cutera Titan work?

Titan utilizes a safe, infrared light to heat the dermis well below the skin's surface. During the procedure, the skin's top surface (epidermis) is protected through continuous cooling with the Titan handpiece.

The Titan Difference

Titan's light-based platform provides a more uniform distribution of energy for heating the deep dermis than other energy sources, such as RF (radio-frequency).

What areas respond best to the Titan procedure?

Patients have been successfully treated on many areas of the body. The most common areas treated are the jaw line, under the chin, and the abdomen.

Who can be treated?

Since the Titan procedure heats tissue under the skin's surface, patients of any skin type can be treated. Men and women of all ages have been successfully treated. Your physician can advise you if the Titan is right for you.

How many treatments will I need?

Although a single treatment has produced very good results for many patients, it is quite common for patients to have two or three treatments over a few months. Your physician will advise you as to the number and frequency of treatments recommended for optimal results.

What happens after the treatment?

Immediately following the treatment, you may notice some redness and mild swelling in the treated area. Both should dissipate within a few hours. Consult with your medical professional for complete information on the benefits and risks of treatment.

What do treatments feel like?

With each Titan pulse you will feel a brief heating sensation. Some patients are more comfortable taking a mild pain reliever beforehand. Many patients are comfortable without any medication. Your physician will advise you.



titan™

“You know you have a winner when patients keep calling to tell you about their great results. With no pain, no downtime, no anesthesia and plenty of results, it is becoming a popular part of my practice.”

Guido Lozada, M.D.
Plastic Surgeon
Honolulu, HI

“Titan has quickly become a very popular procedure in my practice. Patient satisfaction has been very high with over sixty treated patients in just the past three months.”

Ronald Moy, M.D.
Dermatologist
Los Angeles, CA

Schedule your consultation now to learn more about the Titan procedure.

www.titanskin.com